

# QUICK REFERENCE GUIDE

## PACK COMPONENTS

**exaFAT**  
FAT DIALYSING SYSTEM

### FAT INFILTRATION & HARVESTING

- A** CANNULA 3mm x 250mm
- B** SYRINGE VacLok 20ml X2




### FAT PROCESSING

- C** CLAMP
- D** INFUSION LINE
- E** SPATULA
- F** FAT DIALYSING STATION



### FAT INJECTION

- G** CANNULA  
ROUND TIP • 2,1mm x 150mm
- H** CANNULA  
SPATULATED TIP • 2,1mm x 150mm
- I** CONNECTOR FLL-FLL
- J** NEEDLE 16G x 90mm

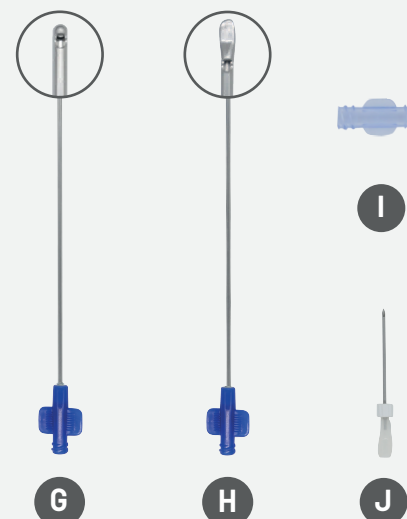

### TISSUE EXPANSION

- K** NEEDLE  
WITH STYLET 16G x 150mm



REV4/2026 REFERENCE: EXAFAT-04

## QUICK REFERENCE GUIDE

# FAT DIALYSING STATION / STEP BY STEP

**exaFAT**  
FAT DIALYSING SYSTEM



### BEFORE DIALYSING THE FAT

1. **Close** the 3-Way Stopcock on the WASTE bag
2. Place the WASTE bag on the floor



**1**

## FILL

1. Close the WASTE clamp (C)
2. Open the infusion line (D) connected to the WASH-IN port
3. **Fill the station with Dialysing Solution** (NaCl 0.9% or Ringer's Lactate) **until the filter is completely immersed**
4. Close the infusion line

**3**

## MOBILISE

1. **Gently mobilise** the lipoaspirate **UP / DOWN** using the spatula (E), until fully homogenized
2. Let rest for up to 10 seconds, until the oil rises to the surface

**5**

## COLLECT

1. Using the spatula, **gather the dialysed fat at the bottom** of the station
2. **Extract** the fat with Luer-Lock syringes while assisting with the spatula

Once all the dialysed fat has been collected, the EXAFAT STATION is ready for the next cycle

**D**

**B**

## LOAD

Transfer up to 100 mL (5 x 20mL) of lipoaspirate through the LIPO-IN port using the VacLok Syringes (B)

**2**


**E**

## DIALYSE

1. Open the WASTE clamp (C)
2. Open the infusion line (D)
3. **Gently mobilise** the lipoaspirate **UP / DOWN, then LEFT / RIGHT**, using the spatula, until no blood / oil remains
4. **Close the infusion line (D)**
5. **Gently mobilise** the lipoaspirate **UP / DOWN, then LEFT / RIGHT**, using the spatula, until no fluid / oil remains
6. **Close the WASTE clamp (C)**

CONTINUOUS WASH

**4**

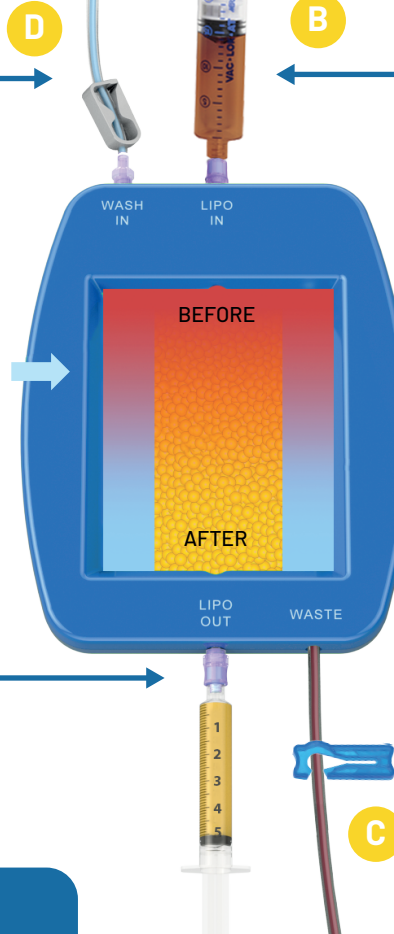

**VIDEO  
STEP  
BY STEP**

